


BUSTER Quick Collar Sizing Guide




Provet Code	Size cm / inch	Example Suitable Breeds	Weight kg
COLL B Q01	7.5cm / 3"	Pomeranian, Chihuahua	1-3kg
COLL B Q02	10cm / 4"	Yorkshire Terrier, Bichon Frise, Maltese	3-5kg
COLL B Q03	12.5cm / 5"	Papillon, Border Terrier	5-10kg
COLL B Q04	15cm / 6"	Scottish Terrier, Beagle, Irish Terrier	10-15kg
COLL B Q05	20cm / 8"	English Springer Spaniel, Staffordshire Bull Terrier	15-25kg
COLL B Q06	25cm / 10"	Labrador Retriever, Siberian Husky, Irish Setter	25-30kg
COLL B Q07	30cm / 12"	Golden Retriever, Boxer, Collie, German Shepherd	25-35kg

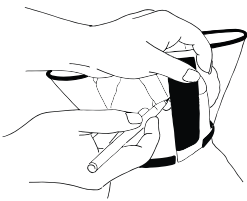

Step 1

- Fold the collar around the neck
- Mark the selected position
- Leave two fingers width between the collar and neck
- Keep the touch fastener mated throughout the sizing procedure

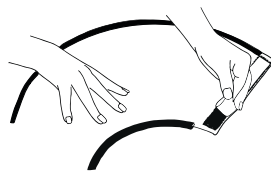


Step 2

- Remove the adhesive liner from the touch fastener



Step 3

- Apply the adhesive part to the marked position
- Cut off redundant collar material

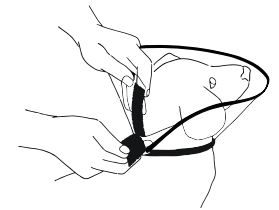

Step 4

- Place the collar on the pet's neck

BUSTER Paw Booties Sizing Guide




BUSTER Bootie Hard Sole

Recommended for dogs with injuries requiring a heavier splint, or a thicker bandage for more than a few weeks.



BUSTER Bootie Soft Sole

Recommended after paw injury or a surgery procedure for a couple of weeks use.

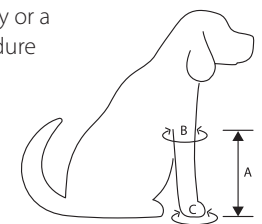

Provet Code Hard Sole	Provet Code Soft Sole	Size	Colour	Height cm/inch (A)	Top circumference cm/inch (B)	Sole circumference cm/inch (C)
PAW B K01	PAW B K08	X Small	Yellow	15.5cm / 6"	20cm / 8"	18.5cm / 7.25"
PAW B K02	PAW B K09	X Small*	Pink	21cm / 8.25"	28cm / 11"	22cm / 8.5"
PAW B K03	PAW B K10	Short	Orange	15cm / 7"	26cm / 11"	25.5cm / 10"
PAW B K04	PAW B K11	Small	Blue	27cm / 10.5"	30.5cm / 12"	26.5cm / 10.5"
PAW B K05	PAW B K12	Medium	Red	33cm / 13"	38cm / 15"	33cm / 13"
PAW B K06	PAW B K13	Large	Green	46cm / 18"	44.5cm / 17.5"	36.5cm / 14"
PAW B K07	PAW B K14	X Large	Purple	58cm / 23"	53.5cm / 21"	40.5cm / 16"

*X-Small (Pink) size features a taller boot than X-Small (Yellow)




A covetrus Company

BUSTER Ear Covers

Sizing Guide



Provet Code	Size	A Circumference around the forehead in cm	B Circumference around the neck in cm	C Length from forehead to neck in cm
EAR C B01	X-Small	20.5cm	24cm	12cm
EAR C B02	Small	24cm	26cm	15cm
EAR C B03	Medium	27.5cm	29.5cm	17cm
EAR C B04	Large	31.5cm	33.5cm	19.5cm
EAR C B05	X-Large	35cm	38cm	21.5cm
EAR C B06	2 X-Large	38cm	42cm	24cm

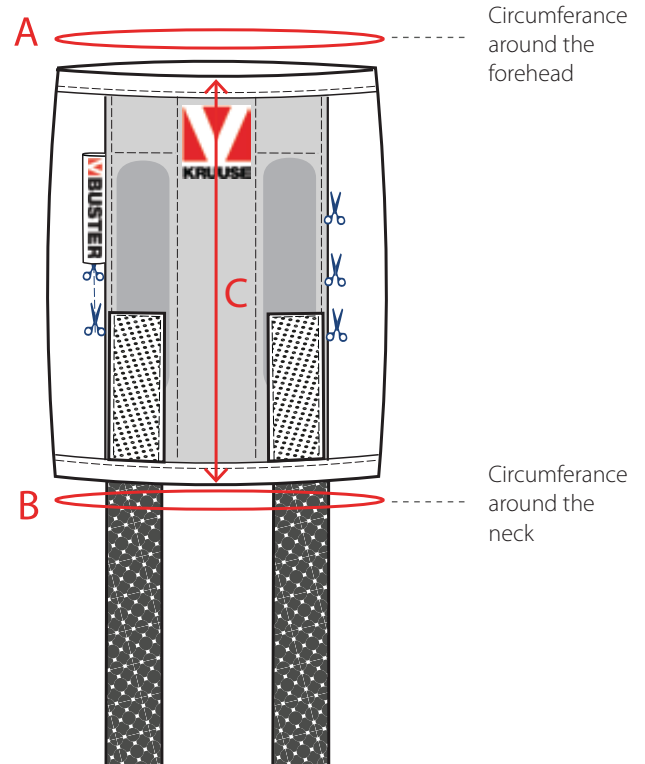

Cut a hole in the non-fraying fabric for the healthy ear flap

To order, contact your local Customer Service Team, Business Account Manager or order online through Provet Connect connect.provet.com.au

